

Is it really a breath of 'fresh air'?

AlcoMate breath alcohol detectors can tell you



The AlcoMate Prestige™ is a professional-grade breath alcohol detector that utilizes an advanced semiconductor-oxide sensor to test breath alcohol content quickly and accurately.

This simple one-button operation provides a digital reading in seconds. At just a fraction of the cost of other DOT-approved units, the AlcoMate Prestige™ is one of the top selling breath alcohol screening devices on the market.

**Lowest-priced
mouthpieces
on the market**

For the AlcoMate Prestige™
and AlcoMate Pro™

Alcohol Content Levels in Males and Females

For Males

Body weight (lbs)	1 drink	2 drinks	3 drinks	4 drinks	5 drinks	6 drinks	7 drinks	8 drinks	9 drinks	10 drinks
100	.043	.087	.130	.174	.217	.261	.304	.348	.391	.435
125	.034	.069	.103	.139	.173	.209	.242	.278	.312	.346
150	.029	.058	.087	.116	.145	.174	.203	.232	.261	.290
175	.025	.050	.075	.100	.125	.150	.175	.200	.225	.250
200	.022	.043	.065	.087	.108	.130	.152	.174	.195	.217
225	.019	.039	.058	.078	.097	.117	.136	.156	.175	.198
250	.017	.035	.052	.070	.087	.105	.122	.139	.156	.173

For Females

Body weight (lbs)	1 drink	2 drinks	3 drinks	4 drinks	5 drinks	6 drinks	7 drinks	8 drinks	9 drinks	10 drinks
100	.050	.101	.152	.203	.253	.234	.355	.406	.456	.507
125	.040	.080	.120	.162	.202	.244	.282	.324	.364	.404
150	.034	.068	.101	.135	.169	.203	.237	.271	.304	.338
175	.029	.058	.087	.117	.146	.175	.204	.233	.262	.292
200	.026	.050	.078	.101	.126	.152	.177	.203	.227	.253
225	.022	.045	.068	.091	.113	.136	.159	.182	.207	.227
250	.020	.041	.061	.082	.010	.122	.142	.162	.182	.202

The Time Factor

Hours since first drink	Subtract this from BAC
1	.015
2	.030
3	.045
4	.060
5	.075
6	.090

A person's actual blood alcohol content (BAC) is dependent on many complex factors, including their physical condition (body composition, health, etc.) and what they have recently ingested, including food, water, medications, and other drugs. The results that are generated are rough estimates of an average healthy person's BAC assuming typical beverage sizes, recipes, and alcohol content. The BAC estimates generated by the above charts should not be used to infer anyone's fitness to work, drive, or perform any other task or duty.

The AlcoMate Prestige™ provides only presumptive results for blood alcohol content. This device is not intended for use to determine blood alcohol content as it relates to any user's fitness for operating equipment, driving, or any other activity.